



## FFP2 Mask

### FFP2 Disposable face mask

25 pcs.

#### FFP2 NR – Model BONNIE


##### Instructions for use

- Wash and spray your hands, fold out the mask and place it over the mouth and nose with the noseband at the top.
- Squeeze the noseband and make sure that the mask closes tightly around the mouth, chin and nose.
- Pull a strap over each ear.
- Test the fit of the mask with a strong inhale. The mask should be sucked in towards the mouth.

mask complies with DIN EN149:2009-08 and the PPE regulation EU 2016/425 and is CE marked no. 2163. The respirator is disposable. Replace it with a new, dry, clean, and well-fitting mask after if it is damp, if it is visibly dirty, or if it does not fit properly. Product shelf life: 3 years when stored at temperatures from -5°C to +25°C and at a maximum humidity of 80%.

NR mark means that the mask can be used for a maximum of one work shift (8 hours).  
re to follow these instructions or misuse can reduce masks or cause illness or permanent

age.

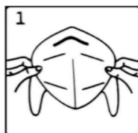

Produceret by: **VISITET A/S**  
Tjørbyvej 90, 8930 Randers  
Kontakt:  
[info@circulamedico.com](mailto:info@circulamedico.com)



CE  
2163

Date: 11.2020  
LOT nr.: 2020112163  
Expiry date: 10.2023

Made in Denmark

